



## CARE ARMREST TOILET ROLL INSTALLATION INSTRUCTIONS PLEASE READ CAREFULLY BEFORE INSTALLATION

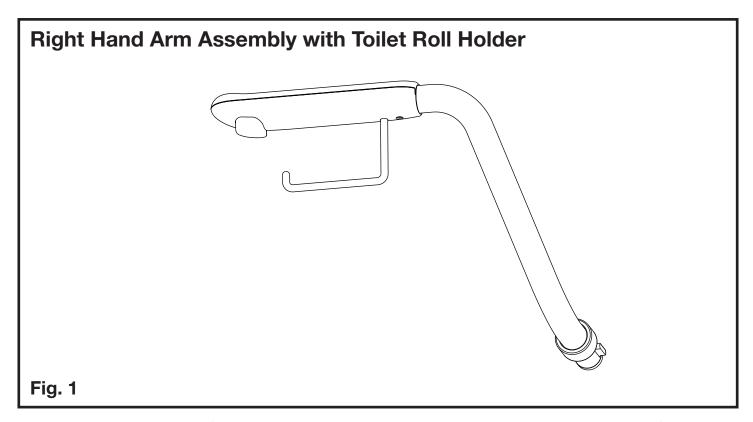


1. Toilet roll holder comes fitted to the right hand side, however can be swapped to the left hand side if required.

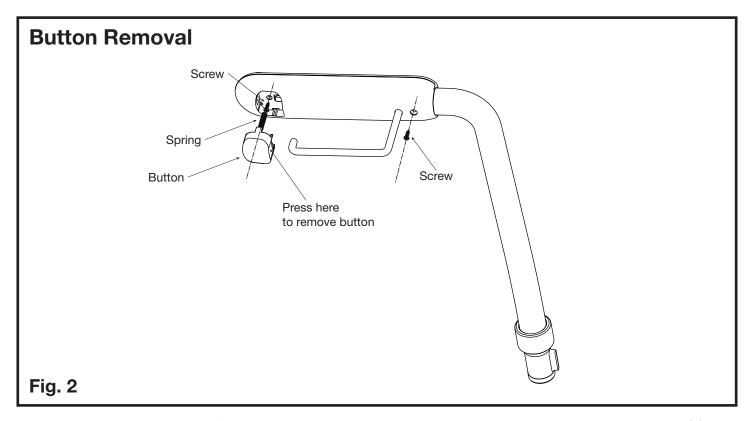


2. Unclip button with small flat bladed screwdriver. Remove button and springs. Remove two (2) phillips head screws.



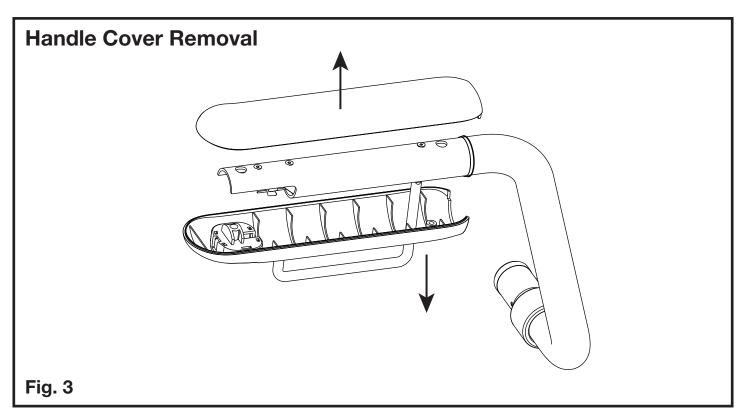

3. Slide lower handle down toilet roll holder. Remove top handle.

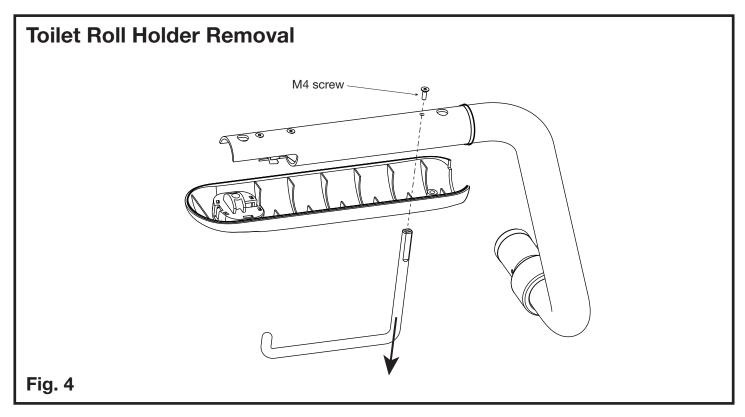

4. Remove M4 phillips head screw and slide out toilet roll holder.

5. To change to left hand side, remove left hand handle and reverse installation instructions.

Date of Issue: 17.01.2013

Issue No: 1 Page 2 of 2